
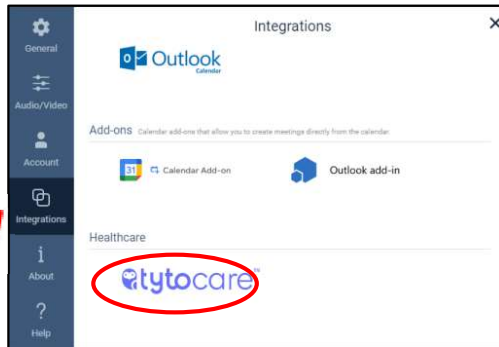





ALLINA VIRTUAL CARE TEAM – TytoCare Device Pairing

1. Go to the VidyoConnect settings cog. 
2. Select Integrations then scroll down and click the TytoCare logo.




3. Click Pair TytoCare device. 
4. Click the skip network details box then click on GENERATE QR CODE.

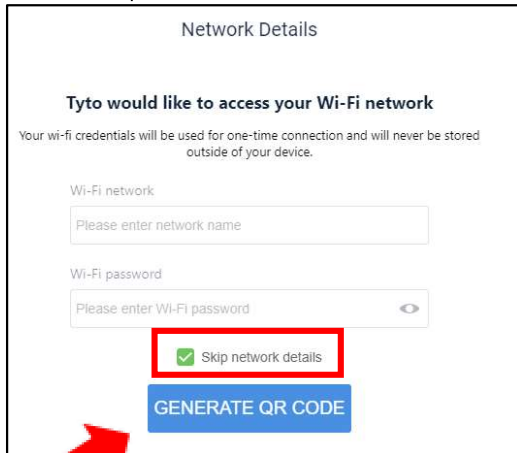

Note: The device should automatically connect to the internet. You can validate by looking at the network icon.



5. The QR code will display.

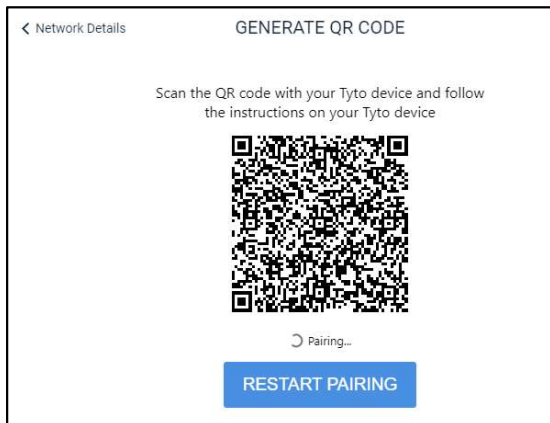

6. Go to steps 7-10 for the Tyto device instructions




ALLINA VIRTUAL CARE TEAM – TytoCare Device Pairing


7. Power on the TytoCare device.



8. Select the three dots, then select **"I have the App."**

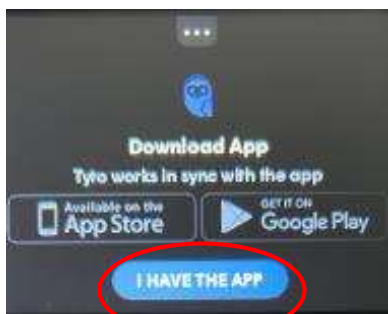

9. Select **Scan QR Code** and scan the VidyoConnect generated QR code (step 5).

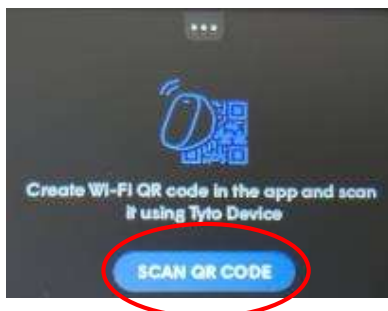

10. Tyto is now paired to the VidyoConnect account/cart. Done!

